


## HOW IT WORKS

Delivers sustained release vapors while compressed nasal flares gently expand your nasal passages to open airways.

## HELPFUL HINTS

- **Proper placement is essential for performance and comfort.** (see fig. 5)
- After removal, you may continue to feel increased airflow for 15-30 minutes.
- As with other products such as new shoes, contact lenses, etc; it is natural to feel tenderness initially. Please be patient during this 1-2 week adjustment period. If necessary, delay use to alleviate discomfort. Resume use after a 24-hr rest period.

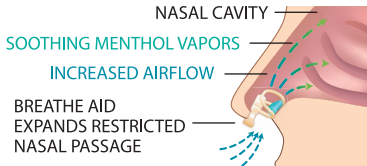

- As your vapor inhaler conforms to your nose, you may feel a light pressure and/or tickle, which quickly subsides.
- The vapor crystal scent is safe and natural. It was formulated to suit the varying tolerances of our customers.
- You may experience olfactory fatigue. The menthol scent may not be noticeable after prolonged day/night use, but it is still present up to 10 days.

**VAPOR CRYSTAL AROMATIC INGREDIENTS:** Camphor, menthol, eucalyptus. **STORAGE:** Store at room temperature. Avoid excessive heat. **CAUTION:** Carefully read, understand and follow directions before use. Using this product other than directed may cause breakage or serious injury. As with all intra-nasal devices, there are inherent risk factors associated with using them. You should proceed with extreme caution while using this product and regularly examine it before and after each use for any defect that may present a potential health hazard. Avoid extreme temperatures. Do not tamper with, or attempt to remove the vapor crystals. Discontinue use if breakage occurs, if the vapor crystals have become loose, or if you have skin or mucosal irritation. Consult your physician before use, especially if you have had surgery or a medical condition relating to the nose, sleep apnea, asthma, immune deficiency, serious respiratory or heart problems; have sensitivity to camphor, menthol, or eucalyptus; or if you are pregnant or breastfeeding. Seek medical attention for abnormal breathing patterns during sleep, daytime sleepiness or difficulty breathing. Do not use for more than 12 hours in a 24 hour period. Not for use if under the age of 18. Persons under the age of 18 must consult a doctor before use. Keep out of reach of children.

# SleepRight

Intra-Nasal  
**Vapor Inhaler**

*Breathe Deep, Sleep Deep*

**Vapor Crystals**  
**Deliver Continuous**  
**Soothing Menthol**

- Colds
- Allergies
- Deviated Septum

**NON-MEDICATED**

**Splintek** *designing health & comfort*

[www.sleepright.com](http://www.sleepright.com) • 1-888-792-0865

©2017 Splintek, Inc. All rights reserved. SleepRight<sup>®</sup> and Breathe Deep Sleep Deep<sup>®</sup> are registered trademarks of Splintek<sup>®</sup>, Inc., 15555 W. 108th St., Lenexa, KS 66219.

U.S. Pats: #7,055,523, #7,461,651, #D652,143,  
#8,048,102, #8,491,622

Not made with natural rubber latex.


This product is manufactured in an FDA inspected facility.



## DIRECTIONS

### Before each use:

- Clean your nasal passages to maximize airflow if desired. Blowing your nose and nasal irrigation may work well.
- Wash your hands.
- Wash your vapor inhaler using cool water for initial cleaning and for fitting lubrication. **Avoid hot water.**

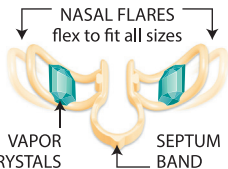


1 Hold the septum band so the nasal flares form the letter *W*.

NOTE: You may prefer to insert your vapor inhaler in the opposite way so the nasal flares form the letter *M*.

You may shape the ends of the nasal flares by gently curling them inward, if more comfort is needed.

NOTE: The septum band can also be formed for more comfort.

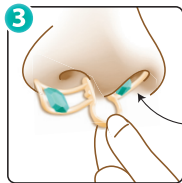

While looking in a mirror, insert one nasal flare into a nostril by tipping the nasal flare in first.

Then use your index finger on the other hand to compress the second nasal flare into the other nostril.




The naturally **ANTIMICROBIAL VAPOR CRYSTALS** inhibit the growth of common bacteria on the device. They do not protect users against bacteria, viruses, germs, or other disease organisms.



While holding the septum band, adjust the vapor inhaler into a **snug position** with the nasal flares resting just inside **the lower cuffs** of your nostrils.

NOTE: It is normal for the septum band to hang below your nose.

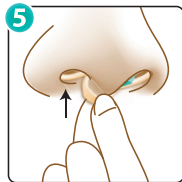

## CARE & CLEANING

Rinse with cool water and dry thoroughly after each use. **Avoid hot water.** Cleaning will not reduce the vapor action. It will last up to 10 days/nights after opening. **Store in the case while not in use to maximize the vapor life span.** Do not tamper with, or try to remove the vapor crystals. Stop use and replace if you notice breakage or weakening.