

**DO NOT BOIL;** ready to use. Eliminates the risk of burns or chemical leaching due to boiling or micro-waving. Not made with BPA, latex, or phthalates.

## RINSE

Rinse with cool water for initial cleaning and lubrication before trying to adjust.  
**Avoid hot water.**

## CHECK HOW IT FITS

Your dental guard has adjustable bite pads, which have been pre-adjusted to fit most people. Place it in your mouth and use a mirror to check how the pre-adjusted position fits before making further adjustments.

The bite pads fit between the upper and lower molars. The front edge of the bite pads should rest between the upper third tooth (cuspid) and fourth tooth (premolar) from the front midline.

The thermal band should rest between your lower lip, cheek and gums. Your body temperature will activate the thermal forming properties of the band for a more comfortable fit. (exclusive feature to SleepRight®)

(Rest teeth on bite surface only, as shown. Never bite the thermal band.)



## ADJUST BITE PADS IF NEEDED




Your dental guard has 5 settings on both sides that allow you to adjust the bite pads forward for a smaller fit and backward for a larger fit, creating sizes XS, S, M, L and XL.

(After initial adjustments, you will not need to adjust your dental guard again.)

Adjustments can easily be made by holding the bite pad firmly with one hand, while gently pushing the band towards the bite pad with the other.

Bite pads should feel tightly secured to the strap. Use water to make adjusting easier.

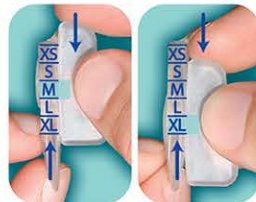

As you rest your teeth on the bite pads, they slightly flex to match the angle of your bite.



Don't pull the band

## HELPFUL TIPS

- For best results, begin wearing 30-minutes before lying down.
- Concentrate on "lips together and teeth apart" to initiate jaw muscle relaxation.
- It is natural to feel jaw muscle fatigue and soreness initially. This is normal due to the natural tendency to clench your teeth. Clenching may even cause minor headaches.

## CLEANING & STORAGE


To clean your dental guard, rinse it with cool water or mouthwash. For further cleaning, brush it with a toothbrush and toothpaste. **Avoid hot water.** Dry thoroughly and store in your case. Replace your dental guard if you notice breakage or excessive yellowing.



Scan for Fitting Video

## HIGHEST BITE PAD QUALITY

Clenching and grinding teeth has been classified as a neuromuscular disorder. Overactive muscles can cause chipping, fractures, excessive wear and loosening of the teeth. SleepRight bite pads fit comfortably between your teeth and interrupt grinding and clenching to protect your smile. We encourage you to relax your jaw muscles during wear. The SleepRight Dental Guard bite pads are made of the most durable materials on the market. SleepRight tests its new and leading innovation in a state-of-the-art research facility so you can have the highest quality and most durable dental guard available.

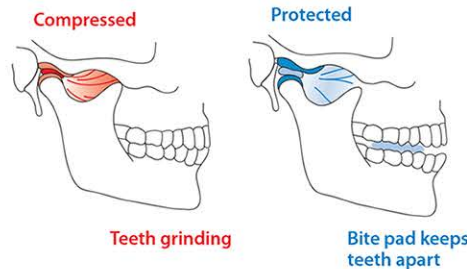

## SPECIAL OFFER

If you are satisfied with your dental guard, please leave a favorable review on the product page from which it was purchased. To show you our appreciation, you will receive a very special offer.

For details visit [sleepright.com/special](https://sleepright.com/special)

**THANK YOU FOR YOUR PURCHASE**

EXCHANGE / RETURN FORM

I am exchanging my dental guard for a:

- ☐ ProRx®
- ☐ Dura-Comfort®
- ☐ Ultra-Comfort®
- ☐ Select-Comfort™
- ☐ Secure-Comfort®

☐ I am submitting a claim for a refund.

Reason for your exchange or return:

\_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_

Name \_\_\_\_\_

Address \_\_\_\_\_

\_\_\_\_\_

Phone \_\_\_\_\_

Email \_\_\_\_\_

My original purchase date was:

\_\_\_\_\_

The LOT# on the bottom of my package is:

\_\_\_\_\_

✂ If you are having problems fitting or adjusting to your dental guard, please feel free to email [support@splintek.com](mailto:support@splintek.com) or call **1.888.792.0865** for assistance; Mon-Fri 8AM – 4:30PM CST. To gain the most success with using your dental guard, read and follow our included **Self-Help Philosophy**.

**1-YEAR LIMITED WARRANTY FOR DURABILITY**  
Splintek, Inc. warrants the durability of your dental guard. You may exchange it for one (1) replacement if you grind through it within 1 year from the original purchase date. This limited warranty does not cover any defects due to misuse, alteration to the product, or negligence. Allow up to 3 weeks to receive your replacement.

**45-DAY MONEY BACK GUARANTEE**  
If for any reason you are not satisfied with your dental guard, you may return it to Splintek for a refund (not including shipping and tax) within 45 days from the original purchase date. Allow up to 30 days to receive your refund.

**Please do not return your dental guard to the store from which it was purchased.**

To submit a claim, complete the included form. Email a photo of the form, dental guard, and proof of purchase to [support@splintek.com](mailto:support@splintek.com). Or, mail these items to the following address at a mailing cost to you.

**Splintek, Inc. Attn: Exchange Processing  
15555 W. 108th St. Lenexa, KS 66219**

The warranty and guarantee are not available for dental guards purchased outside the US and Canada, or from unauthorized sellers, and are limited to the original end-user purchaser. If you have questions regarding whether a seller is an authorized seller, please contact us at the email address or phone number listed above.



**INDICATIONS:** The SleepRight® Dental Guard is indicated for the protection against bruxism or nighttime teeth grinding. It is intended to reduce damage to the teeth and to prevent the noise associated with bruxing or grinding.

**WARNING:** Before first use, read the instructions and safety precautions. Failure to follow these instructions and precautions can result in serious injury or breakage. Regularly examine this product before and after each use for any condition that may create a potential health hazard. **Do not use if you are under 18 years of age;** have TMJ; have teeth or jaw pain; or if the product is broken or damaged. **Do not use as an athletic mouth guard.** Do not use for more than 12 hours in a 24-hour period. Not for use with dentures, braces, or other dental appliances. **Do not use for more than 3 months from initial use without consulting your dentist.** Ask a dentist before use if you have loose fillings, loose caps, loose teeth, chipped or broken teeth, or unfilled cavities; jaw clicking or pain, teeth pain, face pain; difficulty chewing; mouth sores; gum disease or bleeding gums; or serious respiratory problems. **Stop use and see a dentist if you develop loose teeth or a change in your bite;** experience pain or other irregularity; the product easily falls out of your mouth or causes you to gag or choke; you develop bleeding gums, soreness or other reaction inside your mouth. While using this product, see your dentist at least every 6 months.

SELF-HELP PHILOSOPHY

Bruxism is a habit of unconsciously clenching and grinding your teeth. The SleepRight dental guard protects your teeth from clenching and grinding, while the self-help philosophy teaches you to break this neuromuscular habit.

**Awareness**  
The SleepRight dental guard is designed to restore the space that exists naturally between your teeth when you are relaxed. By wearing the SleepRight dental guard 30 minutes before sleep, you will become more aware of the daily clenching and grinding you do during your routines.

**Accountability**  
Take the initiative to break the habit of clenching and grinding by resisting the urge to bite down on the bite pads. Relax your jaw, keep your lips together, and your teeth apart while you are awake. Finding indentations on the bite pads from your teeth means you should make more of an effort to concentrate on "lips together, teeth apart".

**Behavior Modification**  
To transition from clenching your teeth to relaxing your jaw while asleep, you must repeat the mantra "lips together, teeth apart" while awake until it is a healthy habit that no longer needs your conscious attention. Everyone's learning curve is different, so please be patient.

Not made with natural rubber latex  
This product is manufactured in an FDA inspected facility

**Splintek** Innovating Health & Comfort **MADE IN U.S.A.**   
1-888-792-0865 Mon. - Fri. 8AM - 4:30PM CST / [www.sleepright.com](http://www.sleepright.com)  
©2007-2021 Splintek, Inc. All rights reserved. SleepRight®, ProRx®, Dura-Comfort®, Ultra-Comfort®, Select-Comfort™, and Secure-Comfort® are trademarks of Splintek®, Inc.  
U.S. Patents: #5,873,365, #6,152,138, #7,490,609, #8,439,044 REV 10/21

SENSEI